



Suitable light-curing units for the polymerisation of SHOFU veneering composites

Manufacturer	Light-curing unit	SHOFU Universal Pre-Opaque	SHOFU Universal Opaque	Ceramage Pre-curing	Ceramage Final curing	Ceramage UP Pre-curing	Ceramage UP Final curing
Bredent Group	Power Unit 2 LED	60 sec.	180 sec.	60 sec.	180 sec.	60 sec.	180 sec.
Dentsply Sirona	Eclipse Junior	90 sec.	180 sec.	90 sec.	300 sec.	60 sec.	300 sec.
GC Europe	Labolight DUO	1,5 min.	3 min.	-	-	1,5 min.	5 min.
GC Europe	Labolight LV-III	2 min.	3 min.	2 min.	5 min.	2 min.	5 min.
Kulzer GmbH	HiLite Power / 3D	90 sec.	180 sec.	90 sec.	180 sec.	90 sec.	180 sec.
Kulzer GmbH	Heraflash	90 sec.	180 sec.	90 sec.	180 sec.	90 sec.	180 sec.
Kulzer GmbH	Uni XS/ XS	90 sec.	180 sec.	90 sec.	180 sec.	90 sec.	180 sec.
SHOFU DENTAL GmbH	Solidilite LED	30 sec.	90 sec.	30 sec.	90 sec.	30 sec.	90 sec.
SHOFU DENTAL GmbH	Solidilite V	1 min.	3 min.	1 min.	3 min.	1 min.	3 min.
SHOFU DENTAL GmbH	Solidilite EX	1 min.	3 min.	1 min.	5 min.	1 min.	5 min.

Devices not listed are not suitable or not validated. For inquiries about other light curing units, please contact our Customer Service at customerservice@shofu.de.

SHOFU DENTAL GmbH 06/2024